## SORBACT® ROUND SWAB

EN – Instructions for Use

SV – Bruksanvisning

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DA – Brugsanvisning

NO – Bruksanvisning

ET – Kasutusjuhised

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SK – Návod na použitie

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**AU/NZ** Bayport Brands Pty Ltd, 21 Bayport Court, Mornington, Vic 3931, Australia

**LB** Food and Drug Corporation Sal, Byblos Bank building, Rachid Karameh street, Verdun, Beirut P.O. Box 11-8182, Lebanon, Tel. +961-1-862000, [www.fdc.com.lb](http://www.fdc.com.lb)

**IR** NIBYKAN Golden Pharma Mannheim GmbH, Augustaanlage 32, 68165 Mannheim, Germany, Tel. +49 621 43179339

Payk Daru Tosseh, No. 03, Karimkhanezand Ave, 7th of Tir Sq, Tehran, Iran. 1585917114, Tel. +98 21 54549000

**Device Description** Sorbact® Round Swab is a sterile, bacteria and fungi binding wound dressing. It consists of a green Sorbact® wound contact layer shaped to a spherical swab, held together by a silicone ring.

**Mode of Action** Sorbact® binds microorganisms, such as *Staphylococcus aureus* (including MRSA), *Streptococcus species*, *Escherichia coli*, *Pseudomonas aeruginosa* and *Candida albicans*, as shown *in vitro*. These microorganisms are removed from the wound each time the dressing is changed.

**Intended Use** Sorbact® Round Swab is suitable for wound cleansing or as wound filler in management of clean, contaminated, colonized or infected exuding wounds, such as minor cavity wounds.

**Contraindications** There are no known contraindications to the use of Sorbact® Round Swab. Attention is drawn to the warnings and precautions.

### Warnings and Precautions

If the treated condition deteriorates, fails to improve or a side effect is observed, consult a physician or an appropriate health care provider.

Do not use on patients with a known hypersensitivity to any of the dressing components.

The dressing should always be applied in direct contact with the wound area to allow microorganisms to bind to the dressing.

Should the dressing adhere to the wound, moisten or soak the dressing to assist removal and avoid disruption of the healing wound.

Do not use in combination with fatty products, such as ointments, creams and solutions as they may decrease the binding of microorganisms.

Sorbact® Round Swab is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination.

Do not re-sterilize.

Do not cut the dressing.

Do not use if peel pouch is damaged.

Prior to radiation therapy remove Sorbact® Round Swab. A new dressing can be applied following treatment.

Sorbact® Round Swab does not contain x-ray thread and should not be used in open surgery.

Sorbact® Round Swab is MR safe.

**Instructions for Use (wound cleansing)** **1.** Remove the dressing from the pouch using an aseptic technique. **2.** Moisten Sorbact® Round Swab in to the irrigation solution (e.g. saline solution); squeeze out excess. **3.** Gently clean the wound according to local clinical practice.

**Instructions for Use (wound)** **1.** Prepare the wound according to local clinical practice. **2.** Remove the dressing from the pouch using an aseptic technique. **3.** In cavity wounds, apply the dressing in such way to ensure direct contact with the complete wound surface. **4.** Apply an appropriate secondary dressing. **5.** The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days.

**Storage and Disposal** Keep dry and away from sunlight. Disposal should be made according to local environmental procedures.

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